

## Vidar Scanners and Toolkit Overview

### Scope:

OmniPro Accept and l'mRT with Vidar scanner option

### Overview:

Scanner	FW	Toolkit	Windows	ASPI/STI	Notes
<b>VXR-12</b>	<b>3.x</b> <b>4.x</b>	Filedate	98	ASPI	Upgrade FW to 5.05
		1999-10-06	NT		
	-	2000	-	Not validated	
	-	XP	-	Not validated	
<b>VXR-12plus</b>	<b>5.x</b>	Filedate	98	ASPI	Upgrade FW to 5.05
		1999-10-06	NT	ASPI	
	-	2000	-	Not validated	
	-	XP	-	Not validated	
<b>VXR-16</b>	<b>7.x-12.x</b>	Filedate	98	ASPI	Upgrade FW to 13.0
		1999-10-06	NT	ASPI	
		-	2000	-	
		-	XP	-	
	<b>13.0</b>	3.1.9.3	98	ASPI	
			NT	ASPI	
			2000	STI	
			XP	STI	
<b>VXR-16 Dosimetry Pro</b>	<b>7.x</b>	Filedate	98	ASPI	Upgrade FW to 13.0
		1999-10-06	NT		
		-	2000	-	
		-	XP	-	
	<b>13.0</b>	3.1.9.3	98	ASPI	
			NT	ASPI	
			2000	STI	
			XP	STI	
<b>Dosimetry Pro Advantage</b>	<b>40.7</b>	3.1.9.3	98	ASPI	Upgrade FW to 45.5
			NT	ASPI	
			2000	STI	
			XP	STI	
	<b>45.5</b>	4.1.81.0	98	ASPI	Serial no 300485-
			NT	ASPI	
			2000	STI	
			XP	STI	
	<b>46.3</b>	4.1.81.0	2000	STI	Serial no 310000-
			XP	STI	