

Technical Note 997-062_TN001_081222

PrecisePlan RTPS interface

OmniPro Accept version 6.5A

Known issues in PrecisePlan RTPS i/f

I. Conversion not possible

Indication:

When converting more than one energy, there is an error message "It is impossible to create any directory! Conversion is terminated". No conversion of additional energies is possible.

Cause:

Depending on the computer and network security settings, there may be limitations of the file permissions in the default RTPS folder. Full permissions are required to create the RTPS folder structure.

Solution:

In menu Options/File locations... change Current RTPS root path to a folder where all users have full permission. E.g. a shared network disk, a folder at the root (C:\), or a floppy disk. Ensure that the user has full permissions on the selected folder.

II. Wrong file extension on bdopt files

Indication:

When converting beam data optimizing scans the file extension is .bdo instead of .bdopt.

Workaround:

Manually rename the files from *.bdo to *.bdopt after conversion:

Enter the following text into Notepad:

```
rename *.bdo *.bdopt
```

Select All files in Save as type and save the file with the name BDOPT.BAT.

Copy the file into the folder where the *.bdo files are and double-click command file BDOPT.BAT to run the command.

In case of any further questions, please contact us under:

IBA Dosimetry GmbH
Service Department
Bahnhofstrasse 5
DE-90592 Schwarzenbruck
Germany



Service hotline:
phone: +49 9128 607 38
fax: + 49 9128 607 10
mailto: service@iba-group.com