


Technical Note: Mounting LDA-99 Cable Guide


The LDA-99 cable guide is used to mount on the Blue Phantom². It holds the heavy cable from LDA-99 in position.

Mounting procedure:

1. On the top edge of the phantom, locate the cable holder which is also part of axis carrier of the phantom.

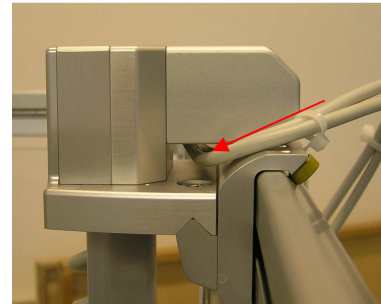


2. (a) Insert the foot (bottom black plastics part of the cable guide) into the hole of cable holder of the phantom.
(b) Tighten the screw to fix the cable guide.

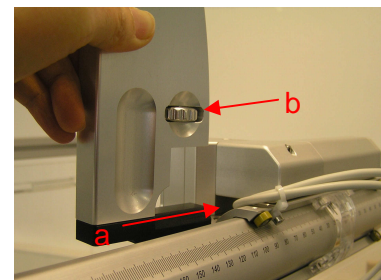

3. Push backwards the upper-right corner to open the ring.



4. Place the cable into the open ring (a) and close the ring (b).

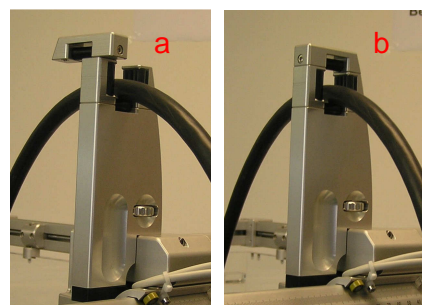



In case of any further questions, please contact us under:

IBA Dosimetry GmbH
Service Department
Bahnhofstrasse 5
DE-90592 Schwarzenbruck, Germany
phone: +49 9128 607 38
fax: + 49 9128 607 10
mailto: service@iba-group.com