


## Technical Note: PH45 Farmer chamber adapter

This manual describes how to handle the new PH45 Farmer chamber adapter for the Blue Water Phantom. It consists of the following parts, to use the original, no watertight Farmer chamber:

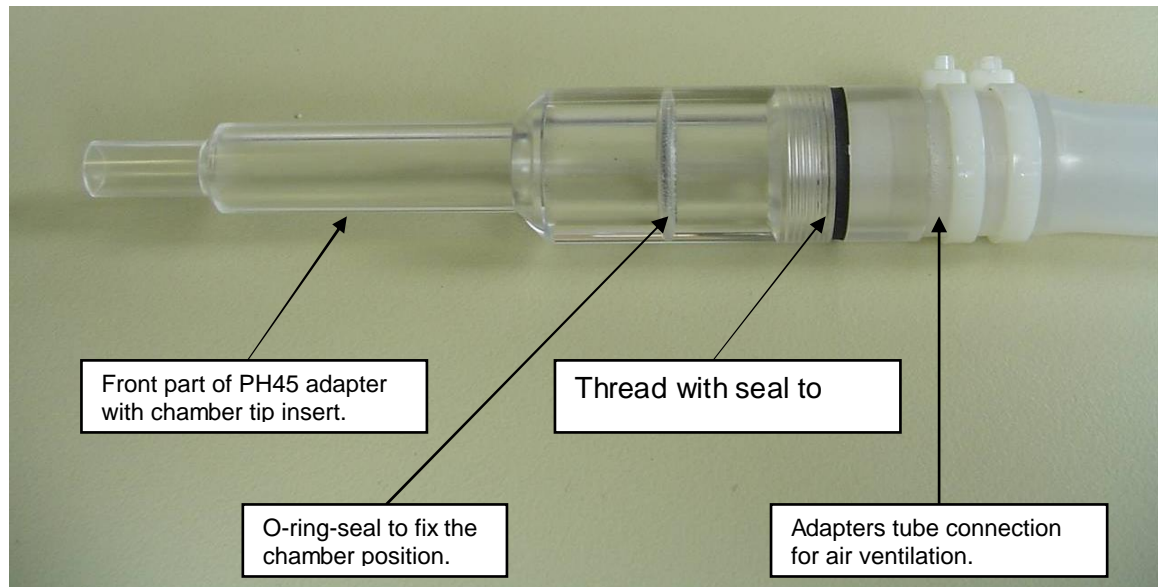

Disassemble the adapter cap and tube by carefully turning the adapter tip counter clockwise (with the left hand) against the fitting where the tube is glued (hold by right hand).



Insert then the Farmer chamber carefully through the silicone tube and fitting.



Insert the chamber into the adapter cap till the chamber tip is at the end position.



Take care that the seal is clean and not broken, otherwise it is not watertight anymore.



Assemble the two parts of the adapter careful by turning the tube fitting on the adapter cap.  
During this, pay attention to the cable to avoid twisting  
Ensure that the chambers end position inside the adapter is not changed during this operation.



To remove the chamber from the PH45 adapter, disassemble the adapter by following the instructions given above in the reverse way.

**Never pull the cable only without disassembling the PH45 - this can damage the chamber.**